



Cleaning Options for micrAgard™ PLUS Wipe Clean Fabric

Thanks to the intelligent design of micrAgard™, the fabric maintains its antibacterial and antimicrobial properties even if it is never cleaned. However, to keep it looking in the best condition possible or to adhere to your organisation's infection control guidelines we have outlined some methods used by some Openhouse customers.

PLEASE NOTE: Use of any cleaning product may cause damage to fabric; these are used entirely at your own risk.

- For removing accumulated dust and grit, use a regular vacuum cleaner or wipe with a clean, damp cloth.
- Any stubborn stains – such as spots of tar – try removing with white spirit. Dab a small amount on with a piece of absorbent cloth or cotton and then wipe it off immediately. Alternatively, try using washing up liquid on a damp cloth.
- Certain chlorine and alcohol based products Clinell Clorox wipes, Actichlor tablets, Steri-7 wipes and isoprenol spray are also suitable for more stringent disinfection if required e.g. in a medical environment.
- Always allow your bag to dry naturally. Do not tumble dry.

IMPORTANT: Always wipe surfaces again with clean water after using any cleaning agent, ensuring no residue is left on the material.

We do not advise using products or methods not listed on this document and we cannot be held responsible for damage caused to bags on which unlisted products or methods are used.

